
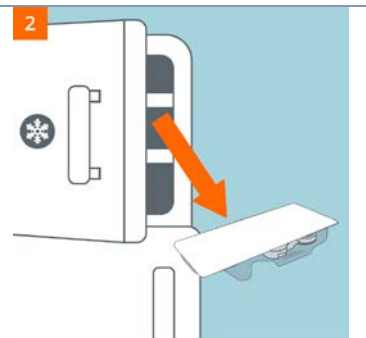
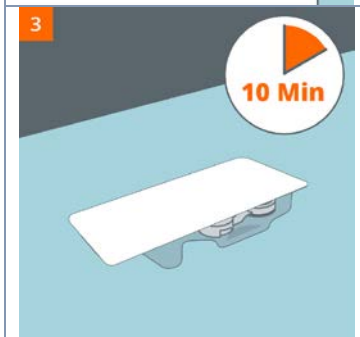
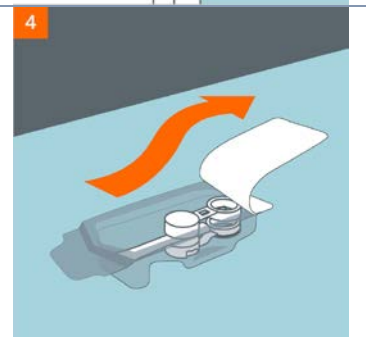
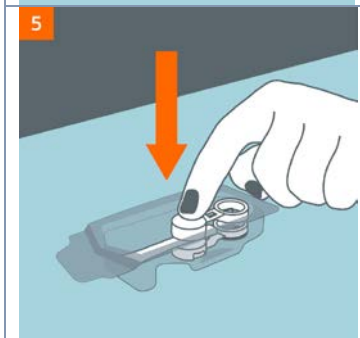

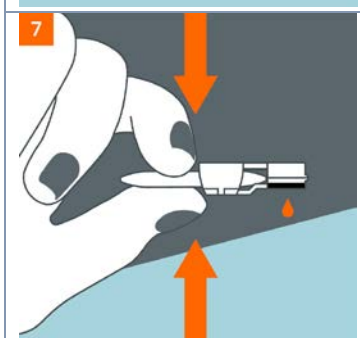

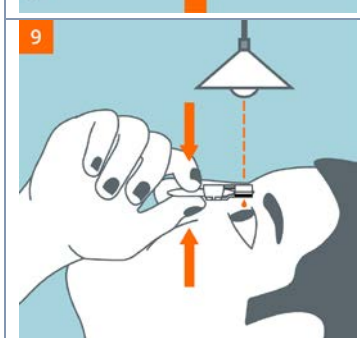




# serum eye drops - instruction for use

	<p>1</p> <p>Wash your hands.</p>		<p>2</p> <p>Get 1 package from the freezer.</p>
	<p>3</p> <p>Let the package stand for 10 minutes.</p>		<p>4</p> <p>Open the seal.</p>
	<p>5</p> <p>Press down firmly on the circular white section in the middle of the device.</p>		<p>6</p> <p>Gently drop the device onto the palm of your hand.</p> <p><i>If the device does not come out of the package, repeat step 5.</i></p>
	<p>7</p> <p>With the dark ring facing downwards, test the device by gently squeezing the tube.</p>		<p>8</p> <p>Position yourself comfortably in a chair and locate a spot on the ceiling; e.g. a lamp.</p>
	<p>9</p> <p>Rest your thumb on your cheek. Look up at the spot on the ceiling and gently squeeze 1 drop into the eye.</p> <p><i>Never let the device touch your eye.</i></p>		<p>10</p> <p>Now squeeze 1 drop into the other eye.</p> <p><i>Never let the device touch your eye.</i></p>
	<p>11</p> <p>After use, dispose of the packaging and the device.</p>	<p><i>You can find further details about the use of the device and an instruction video at: <a href="http://www.mu-drop.nl/how-it-works">www.mu-drop.nl/how-it-works</a></i></p> <p><i>Report device complaints with your pharmacy.</i></p> <p><i>If you have any questions, contact your physician.</i></p>	